

Prophy Paste
Instructions for Use
IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for professional use only.
SPECIALLY FORMULATED FOR USE IN DENTISTRY

IDENTIFICATION

Prophylaxis Paste for the treatment of dentinal and post-operative sensitivity.

Product Name	Prophy Paste
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE **REF**

PP010/0001 – Perfection Plus Medium Mint 250g
 PP010/0002 – Perfection Plus Medium Orange 250g
 PP010/0046 – Perfection Plus Coarse Mint 250g
 PP010/0047 – Perfection Plus Coarse Orange 250g
 PP095/4000 – Eco+ Medium Mint 300g
 PP095/4001 – Eco+ Coarse Orange 300g
 PP010/0051 – Perfection Plus Oil Free Medium Mint 300g
 PP010/0054 – Perfection Plus Oil Free Medium Orange 300g
 PP010/0053 – Perfection Plus Oil Free Coarse Mint 300g
 PP010/0052 – Perfection Plus Oil Free Coarse Orange 300g

FEATURES

Contains no fluoride and only natural flavourings. The blend of abrasive used in the various grades are to maximise the removal of plaque and tartar without appreciable abrasion of the enamel.

INDICATIONS FOR USE

The prophy paste is thixotropic: if desired, the fluidity of the paste can be increased by gentle spatulation before use. Replace lid after use. For best results use with prophy type handpiece and cup/brush combination.

INSTRUCTIONS FOR USE

Fill cup/brush with paste and run at low speed over each tooth surface, making sure that the edge of the cup/brush goes below the free margin of the gingival sulcus. Repeat this procedure for each tooth until all exposed surfaces have been treated. Unwaxed Dental Floss may be used to remove any excess paste through interproximal areas.

STORAGE AND DISPOSAL

Replace lid after use. Store in cool, dry conditions.
 Dispose of in compliance with all local and national regulations, Do not empty into drains.
 Do not reuse empty packaging.


WARNING

Do not expose patients or users known to be allergic to any of the ingredients of this material. Avoid contact with eyes and skin. In the event of contact, rinse with thoroughly with water and seek medical assistance if symptoms persist.


CAUTION

Should a serious incident in relation to the product occur, it should be reported to the manufacturer and the competent authority.

BATCH CODE/EXPIRY DATE

The batch number should be quoted in all correspondence. The batch code gives an open date of manufacture month, year, and day format, with a numerical suffix to uniquely identify the batch of material. Do not use after expiry date.

Perfection Plus Ltd
 6 Westwood Court
 Brunel Road
 Totton
 Hampshire
 SO40 3WX, UK
www.perfectionplus.com



Perfection Plus EU Ltd
 The Black Church
 St. Marys Place
 Dublin, Ireland
 D07 94AX
Regulatory@perfectionplus.com



Advena Ltd
 Tower Business
 Centre
 2nd Floor Tower
 Street
 Swatar, Malta

