

## ProphyPLUS Instructions for Use

### **IMPORTANT**

Read this 'Instructions for Use' document before handling. This product is for use by a dental professional only.  
**SPECIALLY FORMULATED FOR USE IN DENTISTRY**

### **IDENTIFICATION**

Product Name	ProphyPLUS Single Dose pk 200 cups
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

### **PRODUCT CODE - REF**

PP010/0100 ProphyPLUS Single Dose Medium Mint  
 PP010/0101 ProphyPLUS Single Dose Coarse Mint  
 PP010/0102 ProphyPLUS Single Dose Medium Cherry  
 PP010/0103 ProphyPLUS Single Dose Coarse Cherry  
 PP010/0104 ProphyPLUS Single Dose Funky Flavours with 200 stickers

### **INDICATIONS FOR USE**

A blend of polishing and cleaning agents designed for professional use during dental/orthodontic hygiene procedures. Each cup contains enough paste for a single prophylaxis treatment.. The product contains fluoride, it is not considered a substitute for other forms of fluoride treatment.

### **INSTRUCTIONS FOR USE**

1. Using a prophy angle, fill the cup or brush by pressing the face of the cup or brush into the paste while running the angle at low speed.
2. Maintain a quick and light touch so the edges of the cup flair slightly while continuing movement on the tooth. Sufficient paste should be maintained in the cup to avoid overheating of the tooth surface.
3. Evacuation of saliva is recommended to avoid dilution and swallowing of the paste.
4. Rinse mouth with water spray.

### **WARNING!**

Prophy paste should be expectorated and not swallowed.  
 Product contains 1.23% Sodium Fluoride, nausea can result from excessive ingestion of fluoride.  
 Keep out of reach of children.

#### Caution



Should a serious incident in relation to the product occur, it should be reported to the manufacturer and the competent authority.


### **STORAGE**

Store below 25°C. Keep out of direct sunlight. See outer package for expiry date.

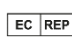


### **Batch code/Expiry Date**

See pot for Lot number and expiration date.

 Dental Technologies  
 6901 North Hamlin Avenue  
 Lincolnwood  
 Illinois 60712, USA  
[www.dentaltech.com](http://www.dentaltech.com)

Distributed by:  
 Perfection Plus Ltd  
 6 Westwood Court  
 Brunel Road, Totton  
 Hampshire  
 SO40 3WX, UK  
[www.perfectionplus.com](http://www.perfectionplus.com)

 Emergo Europe  
 Princessgracht 20  
 2514AP The  
 Hague  
 Netherlands

 Perfection Plus EU Ltd  
 The Black Church  
 St Mary's Place.  
 Dublin, Ireland,  
 D07 P4AX  
[Regulatory@perfectionplus.com](mailto:Regulatory@perfectionplus.com)

